
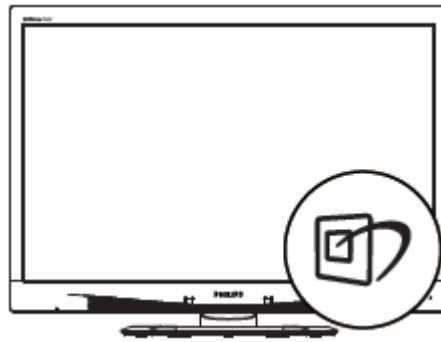

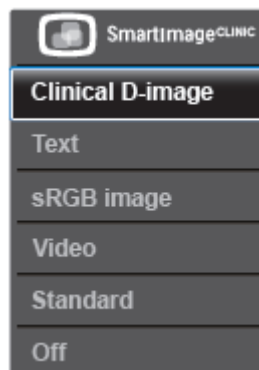










## WoW Monitor Brightness Change Guide

- 1.) Press the  icon on the bottom right of the monitor to launch SmartImage<sup>CLINIC</sup> on screen display



- 2.) There will be six modes to select from. Select Standard and hit /OK on the bottom right of the monitor



- 3.) Press /OK to access the OSD (On Screen Display) menu
- 4.) Press  twice to highlight "Picture"
- 5.) Press /OK and press  once to move the highlighted section to "Brightness"
- 6.) Press /OK
- 7.) Press  or  to the desired setting then press /OK

Created: 30/8 / 2018 - Emerson Cambaya



Western Health